

## Thank you for purchasing Your Personal Portable Oxygen Bar.

I am sure you will be delighted!

This is the basic functioning of this unit. **Your Personal Portable Oxygen** does not create oxygen out of nothing. It creates a cloud of **Oxygen Enriched Air (OEA)** in front of your nose and mouth. It also creates a cloud of “worse” air around the Oxygen Bar – worse meaning containing less oxygen and more nitrogen.

- **Never place the unit directly on a carpet**, but rather on a smooth surface. This allows the “worse” air to escape from around the very bottom. A piece of plywood works, so does tile or bricks – we use clip boards in the office.
- The air compressor tends to get warm with prolonged use. The cooling air escapes from the bottom as well. **Please let the unit cool** for a while after every 30 minute session.
- I recommend that **you use a small fan at lowest speed** setting to keep the air circulating around the unit.
- I recommend using an **outlet strip with switch** – that way the fan comes on automatically every time you use the **Personal Portable Oxygen Bar**.

If you need the headset hose to be longer for working out, etc:

[You can purchase extra hoses from us](#)

- <http://www.drz.org/asp/store/DetailPage.asp?ProductID=936>

**\*\*\*\* Do not exceed a total length of air hose of 13 feet.** More may create too much back pressure for the air compressor.

- Please make sure you do NOT put any undue stress on the “air nipple” on the side of unit. In other words – don’t damage it by suddenly getting up with the head set still on your head.
- **The mouthpiece side is always on left side of head.** The hose exits towards the bottom. The rounded piece of headset rests on ears - the back part acts like a counter weight to the mouthpiece. Aim mouthpiece openings right at the nose/mouth. Try it on yourself first until you get the hang of it before teaching it to others.
- Clean the plastic mouthpiece between uses with grain alcohol, **NOT** rubbing alcohol.

Let me know how that works out for you. E-mail me/call me if you have questions.

# Important Notice for Headset Use

The Oxygen Headset is constructed from plastic material to make it lightweight

- Please treat it with CARE
- Do NOT spread it apart too much - it might break
- GENTLY slide it onto your head
- Be GENTLE when you remove it
- Be CAREFUL where you store it

*Note: The headset is not covered under the warranty*

Dr. Z recommends:

1. **Before using the headset**, please open headset mouthpiece plate with the small clip; **discard the small white foam piece** on the inside.
2. Also, **discard the soft black mouthpiece covers**.

Some sensitive individuals may be allergic/sensitive to the material.

## **!! Change the Personal Portable Oxygen Bar air filter regularly**

- With daily use, you may want to change the air filter at least **every three months**.
- When the air filter gets clogged, the air compressor has to work too hard and gets hot - not a good thing.
- **[You can purchase replacement air filters from us](#)**
- **<http://www.drz.org/asp/store/DetailPage.asp?ProductID=807>**

# **Your Personal Portable Oxygen Bar – Return Policy**

**You have a 30 day money back guarantee** - the unit must be received by us within 30 days from the date we shipped it to you.

**\*\*\* No Returns with International Orders.**

Simply e-mail Dr. Z at [drz@drz.org](mailto:drz@drz.org) to request a **Return Merchandise Authorization Number**.

**RMA requests must be done in writing, by e-mail or by mail, NOT by phone.** We will get back to you as soon as possible with “shipping instructions”.

**After** you receive your **Return Merchandise Authorization Number**, please return your Personal Portable Oxygen Bar for a refund, minus our shipping cost. There is a 15% restocking fee.

**Please note –**

- The unit must be in good working order, in as new condition.
- All accessories must be included.
- Please include ALL original packing material and written materials

Please ship via **UPS** to allow tracking.

**Please insure the return package** for the full amount of your purchase.

A refund check will be issued and mailed to you within 15 business days.

We appreciate your business.

Sincerely

Dr. Z

Peter Zeischegg MS, DC, DACNB

Board Certified Chiropractic Neurologist